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## EN - KELORED CE

**Treatment and prevention of hypertrophic scars and keloids. Improves the appearance of scars and reduces redness and itching.**

Free from fragrances.

Ceramol Kelored is a clear gel containing a blend of silicones with different molecular weights. It improves the appearance and reduces the thickness of scars and keloids which have formed after surgery, traumatic injuries and burns. It restores skin hydration to its physiological level and makes scars softer and smoother. It also helps reduce redness and itching.

**Use:** clean and dry the affected area, then apply a thin layer of product twice a day and massage gently. For best results, use for at least 2 months. Gently remove any excess product to avoid staining clothes. Make-up and cosmetics may be applied after the product has dried. The product is suitable for adults and children.

**Warnings:** do not apply to damaged skin, open or healing wounds, membranes, mucosa, areas treated with topical medicines or near the eyes without first consulting a doctor or pharmacist. In case of reaction due to individual hypersensitivity to one or more of the ingredients, discontinue use and consult your pharmacist or doctor. Avoid contact with the eyes. In the event contact occurs, rinse well with plenty of water. Store away from light and heat sources at a temperature below 30 °C. Close the tube carefully after use and use within 12 months of opening. Consult your doctor before using the product during pregnancy or while breastfeeding. Keep out of reach of children. FOR EXTERNAL USE ONLY.

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Active skin protection.

Ceramol is a range of dermatological products for sensitive, intolerant and allergic skin. Ceramol is grounded on safety, efficacy and skin comfort and delivers innovative solutions to help people look and feel their best. Ceramol products interact with the skin, stimulating its defence and repair mechanisms.

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UNIFARCO SpA - 62, Cal Longa - 32035 S. Giustina - I